



Oxivir Sporicide Wipe

Cleaner and disinfectant for non-invasive medical devices and non-porous hard surfaces

Description

Diversey Oxivir Sporicide Wipe is a one-step cleaner disinfectant wipe effective against *Clostridioides difficile* and *Bacillus subtilis*.

Based on Diversey's patented Accelerated Hydrogen Peroxide (AHP) formulation Oxivir Sporicide Wipe can be used on hard, non-porous surfaces and non-invasive medical devices.

Key properties


- Effective against *C. difficile* in 5 minutes
- Bactericidal, Yeasticidal, Virucidal and Sporicidal in as little as 1 minute
- Positive user and surface safety profile, when compared to other sporicidal products
- Effective Cleaner & disinfectant in 1 product
- Fragrance free
- Active ingredient in AHP breaks down into water and oxygen so no harmful residues left on the surface
- Does not contain chlorine or alcohol

Benefits

- Excellent, quick and effective *C. difficile* outbreak response
- Broad spectrum efficacy against all other pathogen classes provides highest levels of patient protection
- Short contact times keep the surface wet thus ensuring disinfectant efficacy
- No rinsing required
- Does not cause respiratory irritation
- Suitable for use in high risk areas in Healthcare, including ICU, NICU, patient and isolation rooms

Use instructions

1. Wear appropriate gloves (refer to safe handling section on Page 2)
2. Remove gross soils.
3. Open the wipe package, remove single wipe and close the lid.
4. Wipe the surface, assure that the whole surface is moistened, allow to air dry.
5. Ensure surface stays wet for duration of contact time.
6. Change the wipe when is not wetting the surface; visibly soiled or moving to other cleaning area.

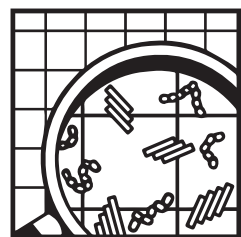

° A.I.S.E.

**This dosage is according to optimal conditions, recommendations may vary, please consult with your Diversey representative for directions.*



Oxivir Sporicide Wipe

Technical data

Appearance: Polypropylene wipe, 25GSM

Wipe Size: 200mm x 300mm

pH value (in use): 2.7 ± 0.3

Relative density (20°C): 1.033

The above data is typical of normal production and should not be taken as a specification.

Safe handling and storage information

Recommended gloves for the use of Oxivir Sporicide:

Robust nitrile powder free.

Chemical resistance to EN ISO 374-1:2016 specification

Microorganism resistance. EN ISO 374-5:2016 specification

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet; sds.diverseysteam.com

Store in original closed containers away from extremes of temperature.

Product compatibility

Gloves should be worn when using Oxivir Sporicide wipe

Microbiological data

Please refer to the EN Table on the last Page

Environmental information

The surfactants used in this product are biodegradable in compliance with the requirement of EU Detergent Regulation, EC 648/2004.

Pack size

Oxivir Sporicide Wipes are available in a Canister of 12x80pc

AHP® and Design and Oxivir® are trademarks of Diversey, Inc.



Oxivir Sporicide Wipe

		Dirty conditions
	EN tests	Contact time
Bactericidal	EN1276*	1 min
	EN13727	30 sec
	EN16615	1 min
Yeasticidal	EN1650*	1 min
	EN13624	1 min
	EN 16615	1 min
Virucidal	EN14476	1 min
	(Adenovirus, Poliovirus, Murine Norovirus)	
Fungicidal	EN1650*	6 min
	EN13624	6 min
	EN16615	7 min
Mycobactericidal	EN14348 Mycobacterium avium	5 min
	EN14348 Mycobacterium terrae	10 min
	EN16615 (Mycobacterium avium, Mycobacterium terrae)	7 min
Sporicidal	EN13704 (B. subtilis)*	15 min
	EN17126 (C. difficile)	5 min
* Tests performed with dirty soiling for non-healthcare areas		