

Oxivir Excel Wipe Acti

Cleaner-disinfectant wipes for non-invasive medical devices and nonporous surfaces

Description

Oxivir Excel Wipe Acti, powered by AHP technology, is the ideal product choice for one step cleaner disinfectant One ready to use wipe for all areas of use to clean and disinfect hard surfaces as well as for non-invasive medical devices surfaces (for professional use only).

Key properties

- Ready to use wipe, ideal product for all areas of uses including HealthCare areas, non-HealthCare areas and food sectors
- Full virucidal
- Fast kill claims for a broad spectrum of viruses, bacteria and yeasts
- Superior Cleaning with high-performance surfactant system
- Fragrance free
- No rinsing required
- Active break down into water and oxygen
- BPR approved biocide
- Certified medical device

Benefits

- Oxivir Excel Wipe Acti provides broad spectrum efficacy against bacteria, yeasts and viruses in short contact times providing the highest levels of disinfection against multiple pathogens of concern
- BPR approved and extensively tested against European Norms
- Oxivir Excel Wipe Acti carries a full virucidal claim including proven efficacy against non-enveloped viruses such as Polio, Adeno, Murine Norovirus and enveloped viruses such as Coronavirus, Vaccinia virus, HIV, HBV, HCV and Influenza
- Oxivir Excel Wipe Acti has a strong cleaning performance due to the powerful surfactants which make it not only an excellent disinfectant but also an excellent cleaner
- AHP formulation is gentle on surfaces extending equipment life and leaving no harsh residues on the surface
- Oxivir Excel Wipe Acti is suitable for daily use cleaning & disinfection hard surfaces as well as for use in high touch areas in retail, education, foodservice areas, long term care and other sectors as well as high risk areas in hospitals such as OR, ICU, NICU & patient rooms
- No PPE requirement, unless for long use



BPR
APPROVED

 **Diversey Europe**
Operations B.V.
De Corridor 4,
3621 ZB Breukelen,
The Netherlands.
www.solenis.com



CE
0044

Oxivir Excel Wipe Acti

Use instructions

Ready to use wipe, no dilution needed, use at room temperature:

- Remove gross soils.
- Open the lid, take a single wipe and close the lid.
- Apply the product by fully wetting the non-porous surface and/or non-invasive medical device.
- Allow to act for 5 minutes.
- Let it air dry. No rinsing of surface required.
- Wash hands after handling.

Application rate: 1.5 m² per wipe.

Technical data

Appearance:	White wipe
Odour:	Neutral
pH value (neat):	<2
Shelf life:	2 years after production (Expiry Date: see packaging) 28 days after opening
Active substance:	0.36% Hydrogen Peroxide

The above data is typical of normal production and should not be taken as a specification.

Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet; sds.diversey.com.

Handle following good industrial hygiene and safety practices. Keep away from food, drink, and animal feedstuff. Do not mix with other products unless advised by Diversey. Store in original closed containers away from extremes of temperature. The product is not intended to be re-used. Only for professional users/specialists.

Warning

Store within the following temperature range: 6°C - 35°C. Do not use if package is damaged. Keep away from sunlight. Handle following good industrial hygiene and safety practices. Keep away from food, drink, and animal feedstuff. The product is not intended to be re-used. Warning: May be corrosive to metals.

Personal Protective Equipment

Full guidance on PPE for this product is provided in a separate Safety Data Sheet; sds.diversey.com.

PPE: Use personal protective equipment as required. **Hand protection:** Rinse and dry hands after use. For prolonged contact protection for the skin may be necessary.

Disposal

Full guidance on disposal of this product is provided in a separate Safety Data Sheet; sds.diversey.com.

Oxivir Excel Wipe Acti

Product compatibility

Oxivir Excel Wipes Acti is intended for use in a hard surface disinfectant incl. hard surface of non-invasive medical devices. Under recommended conditions of use, Oxivir Excel Wipes Acti is suitable for use on surfaces such as stainless steel, chrome, vinyl, nylon fabrics, laminated surfaces, glass, rubber, hard and flexible plastics (polypropylene, polyurethane, polyethylene, PVC acrylic, fiberglass and polycarbonate). Uses other than those identified are not recommended. Oxivir Excel Wipes Acti is not intended to be used on invasive medical devices. **Do not mix with other products. Not intended for porous or acid sensitive surfaces.**

Microbiological data

Area of use : PT2 Healthcare in dirty conditions, PT2 Non Healthcare in dirty conditions & PT4 Food contact surfaces in dirty conditions

Application	Claims	EN Norm	Organisms	Dilution	Contact time
Wiping	Bactericidal	EN 13727 EN 1276 EN 16615	E. hirae, P. aeruginosa, S. aureus, E. coli.	RTU	5 min
	Yeasticidal	EN 13624 EN 1650 EN 16615	C. albicans		
	Virucidal	EN 14476	Adenovirus, Murine norovirus, Poliovirus		

Environmental information

The surfactants used in this product are biodegradable in compliance with the requirement of EU Detergent Regulation, EC 648/2004.

Approvals

- Biocide: BPR EU-0032483-0003 1-3
- Medical Device Notified Body: CE0044

www.diversey.com

© 2025 Diversey, Inc. All Rights Reserved. Rev.22/12/2025 en (K02979)