



Oxivir Plus Spray

Cleaner and Disinfectant for non-invasive medical devices and nonporous surfaces. For food and non-food surfaces

Description

Diversey Oxivir Plus Spray is a ready to use cleaner disinfectant for use on hard, nonporous surfaces and noninvasive medical devices.

Key properties

- Bactericidal, Virucidal and Yeasticidal.
- Patented Hydrogen Peroxide based acidic formulation
- Short contact times
- Effective Cleaner
- Fragrance free

Benefits

- Extensively tested against European norms and effective against multiple pathogens of concern, including Coronavirus, HIV, HBV, HCV, Avian Influenza, Pseudomonas aeruginosa, Staphylococcus aureus, Salmonella enterica, MRSA, VRE and Norovirus
- Cleans and disinfects in one step, reducing working time
- Excellent cleaning performance, including descaling and lime limitation removes the need for a pre-cleaning step
- Good user safety profile, with no hazard warnings on RTU format; does not contain respiratory irritants
- AHP formulation is gentle on surfaces extending equipment life
- Good environmental profile. Active breaks down into oxygen and water
- Suitable for use in high risk areas in Healthcare, including OR, ICU, NICU and patient rooms

Use Instructions

Spray cleaning and disinfection:


1. Remove gross soils.
2. Apply to surfaces by either spraying the wiping cloth and wiping surfaces or applying directly to the surface through use of a trigger sprayer.
3. After the contact time, either wipe dry or allow to air dry.

After use, surfaces that come into contact with food, should be rinsed.

Dosage*

Ready to use, no dilution required.

**This dosage is according to optimal conditions, recommendations may vary, please consult with your Diversey representative for directions.*

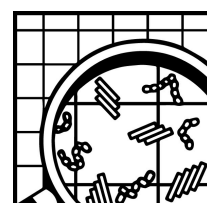

© A.I.S.E



Oxivir Plus Spray

Technical data

Appearance: Clear colorless liquid
pH value (neat): < 2
Relative density (g/cm³; 20°C): ≈ 1.00

The above data is typical of normal production and should not be taken as a specification.

Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet; sds.diversey.com. Store in original closed containers away from extremes of temperature. Only for professional users / specialists.

Warning

Reacts with oxidisable substances.

Product compatibility

Important: Do not mix with other products. Do not use on acid-sensitive surfaces. Under recommended conditions of use, Diversey Oxivir Plus Spray is suitable for use on most materials commonly encountered.

Environmental information

The surfactants used in this product are biodegradable in compliance with the requirement of EU Detergent Regulation, EC 648/2004.

Available pack sizes

Diversey Oxivir Plus Spray is available in 6x0.75L.

AHP® and Design and Oxivir® are trademarks of Diversey, Inc.

Claim table

			Dirty conditions
Claim	EN test	Organisms	Contact Time
Bactericidal	EN 1276*	-	30 sec.
	EN 13727	-	5 min.
	EN 13697*	-	5 min.
	EN 16615	-	5 min.
Fungicidal	EN 1650*	-	15 min.
	EN 13697*	-	15 min.
	EN 13624	-	15 min.
Virucidal	EN 16777**	<i>Vaccinia virus</i>	5 min.
	EN 14476**	<i>Adenovirus</i>	5 min.
	EN 14476**	<i>Poliovirus</i>	5 min.
	EN 14476**	<i>Murine Norovirus</i>	30 min.
	EN 14476**	<i>Vaccinia virus</i>	10 min.
Yeasticidal	EN 13697*	-	5 min.
	EN 16615	-	15 min.
	EN 1650*	-	15 min.
	EN 13624	-	15 min.

* Tests performed with dirty soiling for non-healthcare areas

** Under clean conditions full virucidal activity is reached in 5 minutes.

www.diversey.com

© 2024 Diversey, Inc. All Rights Reserved. 12/02/2024 en (K00240)